VagiQUICK®

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EINFÜHRUNG

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For self-testing

INTRODUCTION

VagiQUICK® is a qualitative rapid diagnostic self test for the detection of *Candida* antigens in vaginal secretions sampled by a swab. The kit is to be used by women who suffer from vaginal symptoms.

The test is based on the immunochromatographic technology. The sample is added to the sample pad with the dilution buffer. When applied to the sample pad, the sample will migrate into the conjugate pad containing anti-*Candida* antibodies, conjugated to blue-colored latex beads. The newly formed *Candida* antigen-antibody-conjugate complex moves onto the nitrocellulose membrane strip. The formed complex is captured on the support membrane by immobilized anti-*Candida* antibodies, producing a signal line. A blue line will appear in the test line region (T) on the membrane. The coloring is dependent on the concentration of the analyte. Any excess of tagged conjugate continues to migrate along the membrane to the control line where it will be captured. This results in the formation of the control line (C), indicating proper function of the test. After 10 minutes, *Candida* concentrations starting from $6,0 \times 10^3$ cells/mL can be detected.

To determine and interpret the results correctly, it is necessary to carefully read the instructions. All details should be understood before performing the test.


TEST CONTENTS

- 1 test cassette in a sealed pouch
 - 1 sterile swab
Copen Italia S.p.A.
Via F. Perotti, 10
25125 BRESCIA, Italy
 - 1 instruction pamphlet
- Additionally required:**
- 1 timer

4. Remove swab from the wrapper.
5. Insert the swab ca. 2 cm (ca. one inch) into the vagina. Now turn the swab around for 20 seconds in order to receive samples of different areas of the vaginal wall. Then remove the swab carefully from the vagina.
6. Dip the swab into the solution inside the purple sealing cap and stir for 20 seconds. This mixes the vaginal secretion with the liquid in the sealing cap.


7. Pull out the swab carefully while squeezing it against the inner wall of the sealing cap. Discard the swab. Do not leave the swab in the device for the following steps.
8. Now hold the test-cassette with one hand by the pad and turn the sealing cap with the other hand in direction of the arrow, anticlockwise all the way to the stop (see illustration a). The indentations of the upper and lower part must exactly align with each other (see illustration b). Now turn the sealing cap back to its starting position (a). This is important for the test to start running.
9. Repeat step 8 two more times. **In its final position, the purple sealing cap has to be in position (b).**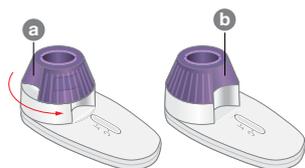

10. Read results after 10 minutes. Occasionally, only a faint test line (T) is visible in case of a positive result. In this case, wait for another 10 minutes before reading the result again.

Do not read the result after more than 20 minutes.

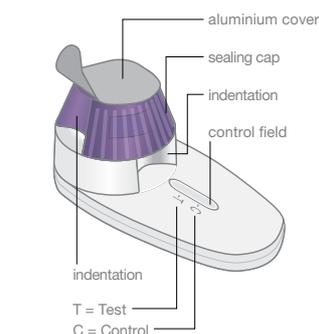TEST PREPARATION

Let test cassette stand at room temperature (15°C to 27°C) before performing the test.

TEST PERFORMANCE

1. Remove the test cassette from the sealed pouch and place it on a dry and level surface.
2. Make sure that you have a timer available.
3. Hold the test cassette with one hand by the pad and remove the aluminium cover from the sealing cap with the other hand.

ATTENTION: The purple sealing cap contains a liquid that must not run out!

EVALUATION OF TEST RESULTS

For the interpretation of results, first determine whether there is a colored line in the test window at position (C). The intensity of the control line is of no significance.

POSITIVE TEST RESULT

If a faint light to dark blue Control line (C) is visible in the result window **along with a light to dark blue Test line (T)**, the test result is positive.

The test result means that ***Candida* antigen is present in the vaginal secretion**. You should see a doctor to confirm the diagnosis and start treatment.

NEGATIVE TEST RESULT

If **only a light to dark blue Control line (C)** is visible in the result window with **no Test line (T)**, the test result is negative.

The test result means that **no *Candida* antigen is present in the vaginal secretion**.

INVALID TEST RESULT

If there is **no Control line (C) or only a Test line (T)** in the result window, the test did not run correctly and the results are not valid.

It is important that you carefully followed the instructions for the test. You should test again with a new sample and a new test.

Explanation of symbols:

Follow instructions	In vitro diagnostic medical device (for external use)	Best before (See imprint on package)
Store at 2°C - 30°C Do not Freeze.	Content sufficient for 1 test	Do not reuse
Manufacturer	Sterilization by irradiation	Batchnumber (See imprint on package)
Catalogue number		

CE0483
Instructions English
Revision from 2016-07 (Rev. 00)

NanoRepro AG
Untergasse 8
D-35037 Marburg
www.nano.ag

NanoRepro
For the love of life.