EN Visit durex.com for a better experience
Durex Fun Explosion Variety 40 condoms

World's No.1 condom brand. With 90 years of experience, Durex is trusted by millions of lovers across the world every day.

Durex Quality. Durex doesn't just meet worldwide condom quality standards, it exceeds the minimum requirement, giving you the confidence to love the sex you have.

Smells better. The special way we make our condoms means they smell better.

This product includes condoms of multiple width and shape. You should use the condom that you feel is securely in place and comfortable based on the size that fits you the best. No method of contraception can give you 100% protection against pregnancy, HIV or sexually transmitted infections. Dermatologically tested. 100% electronically tested.

Using Durex Condoms:

Put a condom on before the penis touches the other person. This helps to prevent pregnancy and the possibility of catching sexually transmitted infections. Check the expiry date on the condom wrapper before you use it. If the individual container is obviously damaged, discard that condom and use a new one from an undamaged package.

Applying the condom:

- Handle the condom carefully, as it can be damaged by fingernails and sharp objects.
- With the roll outside, squeeze the teat and place the condom on the top of the erect penis.
- Roll down to its base. Stop and check if you feel the condom slipping off or tightening excessively onto the penis because this might lead to breakage. Soon after ejaculation the condom should be taken off. The condom should be held firmly in place at the base of the penis before withdrawal.
- Dispose in a bin. Do not flush down the toilet.

Usage advice:

Use only lubricants recommended for use with condoms. Oil-based ones (petroleum jelly, baby oil and some pessaries) and some topical medicines that are applied to the penis or vagina can damage condoms. **STOP USE IF:** you feel discomfort or irritation while using condoms. This condom is made of natural rubber latex, which might cause allergic reactions including anaphylactic shock if the user is allergic to latex. For anal sex use additional lubrication (do not use Durex Tingle or Embrace) on the outside of the condom. If you use a condom for oral sex first, you should use a new condom for any other type of sex that follows. Use a condom only once. Reuse may increase risk of failure or infection.

Consult a doctor or pharmacist if:

you are using topical medicines applied to penis or vagina; irritation/discomfort persists after use; you are unsure if you are allergic to latex; a condom leaks or bursts during use (do this as soon as possible and within 72 hours).

Carelina: UK 0333 2005 345; ROI 01 630 5429

Rev. 02 05/16

Durex and the Durex Logo are trademarks of the Reckitt Benckiser Group of Companies.

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Rev. 02 05/16

DUREX and the DUREX Logo are Trade Marks of The Reckitt Benckiser Group of Companies.

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